





Dispenser Tower

500ml / 1L

- 01- General Features
- 02- Notes and package list
- 03- Bottle placement/exchange, handling and maintenance

Dispenser Tower 500ml/1L

General Features

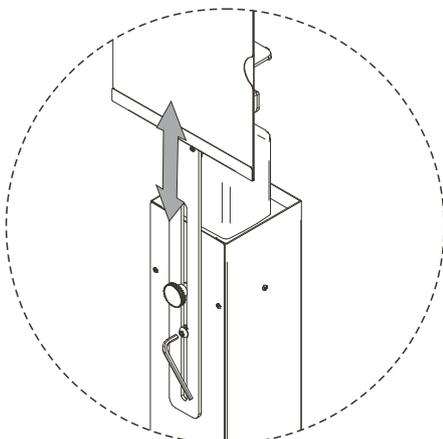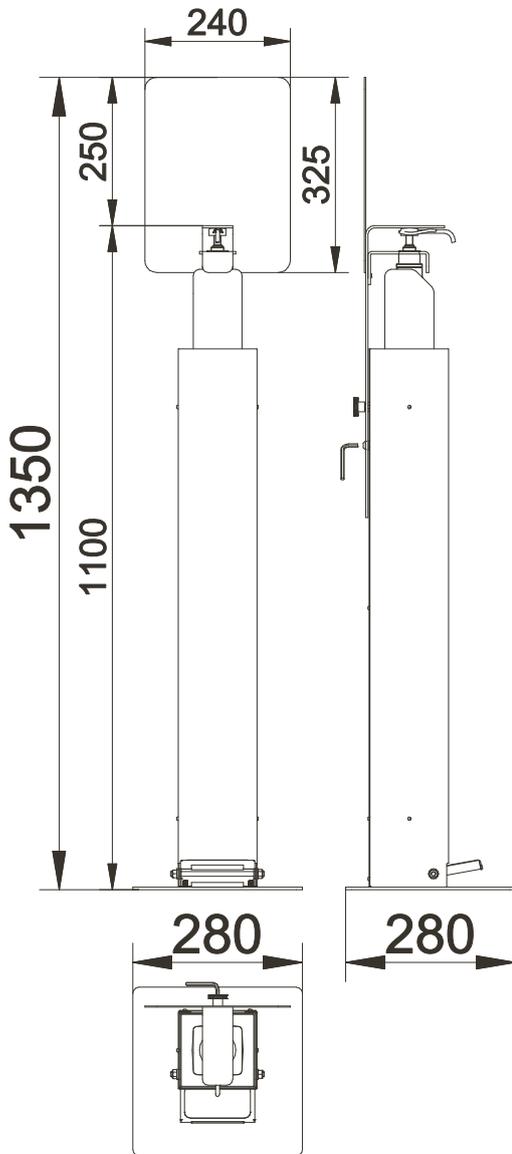

BACK DETAIL

Dispenser Tower operating by stepping a foot pedal.

No need of hands contact.

Compact dimensions.

Mechanic operated. No power supply needed.

Adjustable to 500ml and 1L bottles.

Technical Specifications

DIMENSIONS

Height	Lenght	Width
1100 mm	280 mm	280 mm

WEIGHT

10 Kg. (Disinfection not included).

MATERIAL

ELEMENT	MATERIAL	THICKNESS
Base	Iron	4 mm
Adjustable Piece	Iron	4 mm
Main Body	Iron	1,2 mm

FINISHING

- Colored Epoxy Resin.

Admisible measures for solutions dispensers:

- 975ml / 1L bottle

- 475ml / 500ml bottle

PACKAGE DIMENSIONS

Height	Lenght	Width
1015 mm	297 mm	297 mm

PACKAGE WEIGHT

10,4 Kg. (Disinfection not included).


Dispenser Tower 500ml/1L

User Manual.

Notes:

- Please, read manual carefully before using the Dispenser Tower.
- Dispenser Tower can be used for children under adult supervision.
- Do not use the Dispenser Tower out-doors.
- This Dispenser Tower is suitable for BODE/HARTMANN 975ml/1L and 475ml/500ml bottles.
- It is the installer's responsibility to ensure placement of Dispenser Tower complies with applicable codes, standards and regulatory requirements. Do not place this Dispenser Tower near sources of ignition/flame.


Package include:

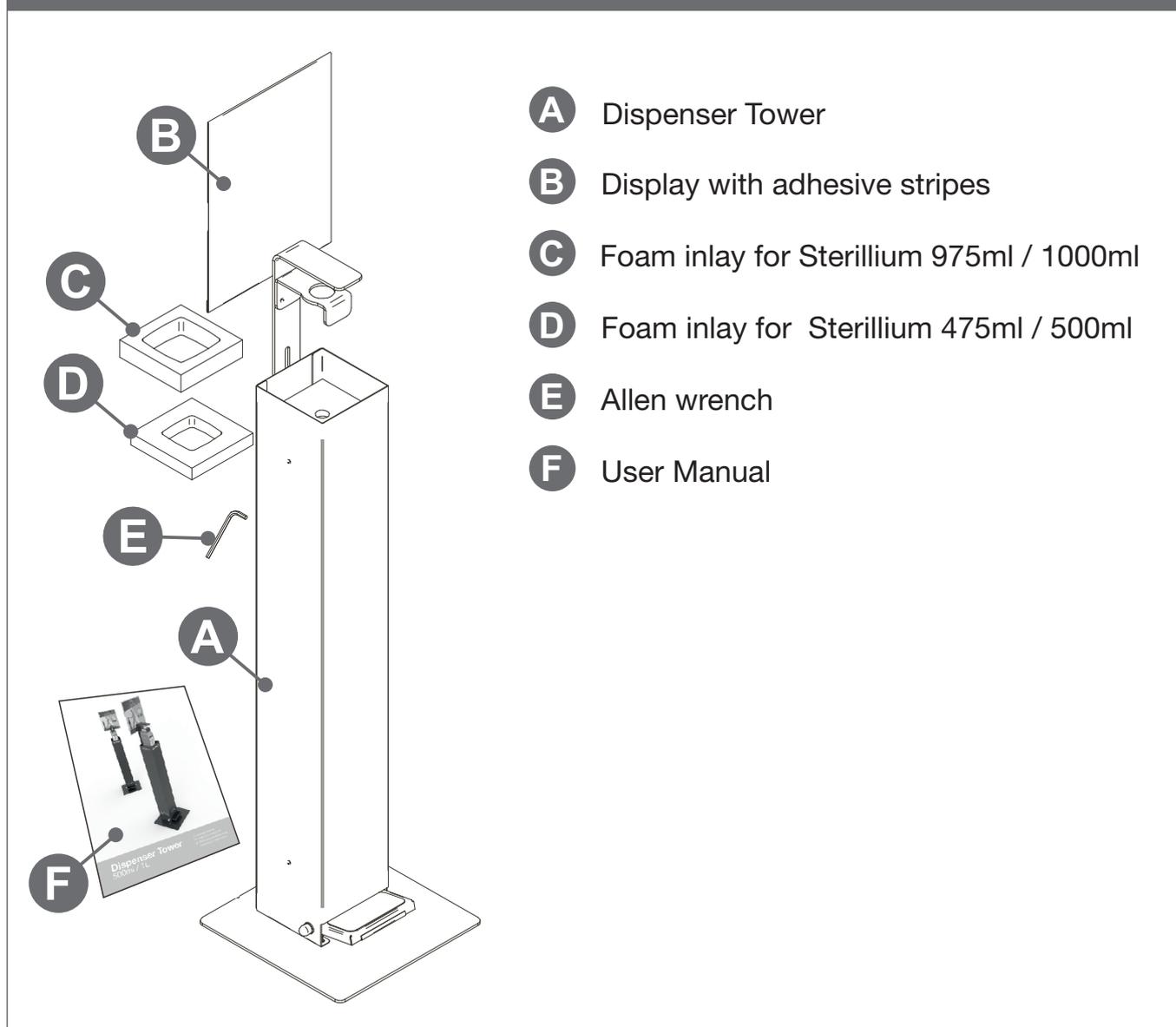

Dispenser Tower 500ml/1L

User Manual.

Bottle Placement/Exchange:

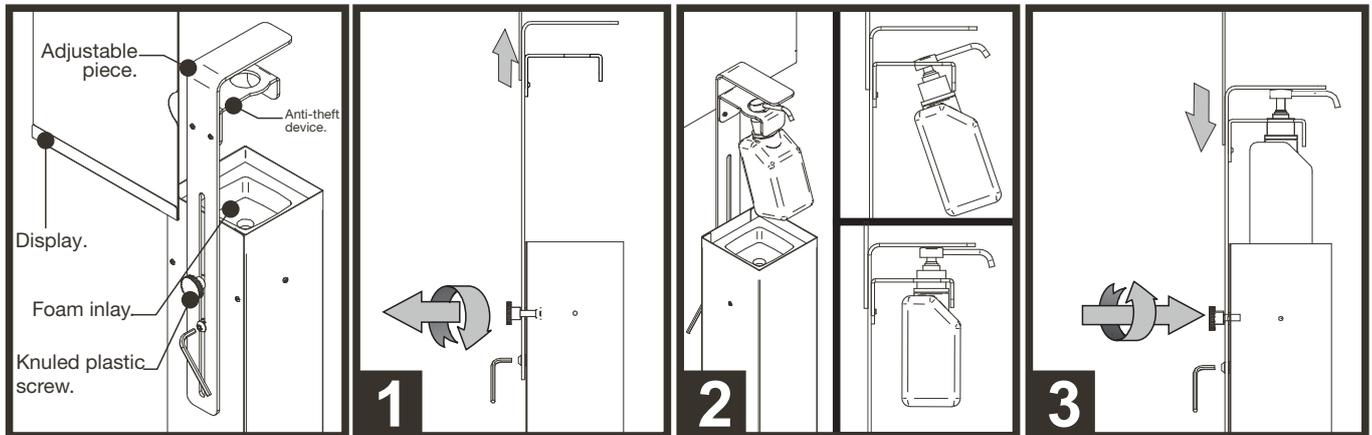

- Unscrew the knuled plastic screw placed on the back until the adjustable piece can be moved up and down. Also unscrew the rounded metal screw placed below if it's needed. (Figure 1)


- Lift up the adjustable element and pass the bottle through the anti-theft device with the pump screwed on. (Figure 2)

- Move down the adjustable piece with the bottle until the bottle is placed in the foam inlay (use the proper foam inlay that fits with the size of the hand disinfection bottle). Place the adjustable piece on top of the pump and lock the adjustable piece by screwing the knuled plastic screw (and rounded metal screw placed below if it's needed for anti-theft protection). (Figure 3)

- The display can be attached to the adjustable piece by using the adhesive stripes.

HANDLING:

- Place your hands under the pump nozzle and then press the foot pedal located at the bottom of the Dispenser Tower. Rub in the hand disinfection for at least 30 seconds.

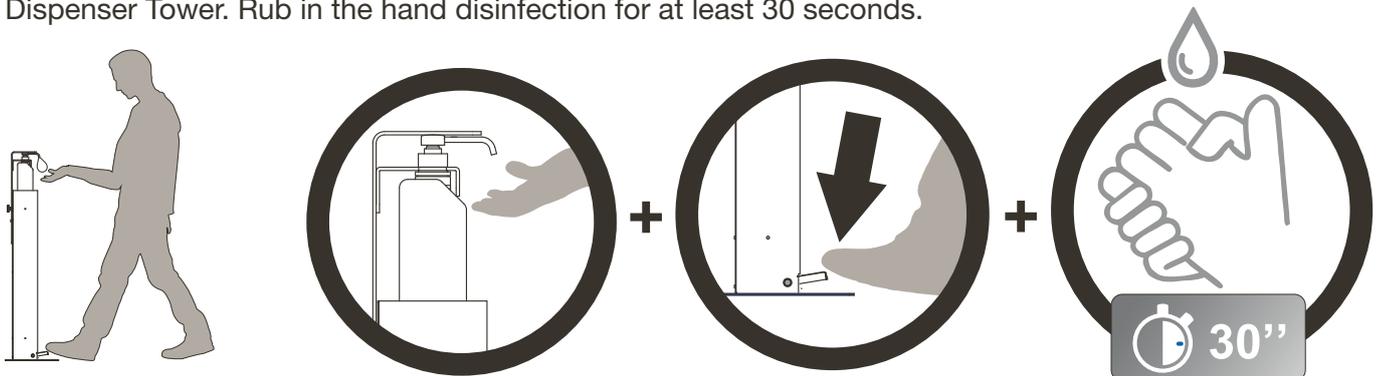

MAINTENANCE:

- It's recommended to clean the Dispenser Tower surface with any of the following BODE products:
 - Bacillol 30 Foam / Tissues.
 - Mikrobac Tissues.
 - Dismozon plus (0,4%).
 - Mikrobac forte (0,5%).